
---

## Bluetooth Smart Scale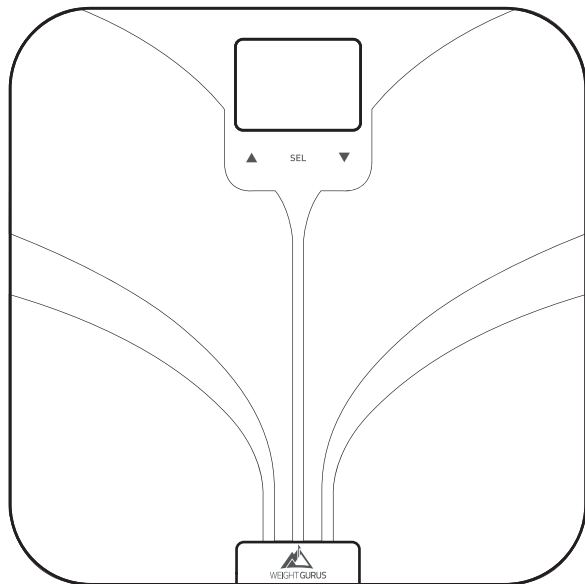

---

**Get Started:** [greatergoods.com/start](http://greatergoods.com/start)

---

CUSTOMER SUPPORT

**Email:** [info@greatergoods.com](mailto:info@greatergoods.com)

**Phone:** (866) 991-8494

**Website:** [greatergoods.com](http://greatergoods.com)

# Table of Contents

Important Safety Notes	2
Introduction	3
Scale Description	4
Physical Features	
Measuring Units	
Setting the Measuring Unit	
Display	
Things To Know Before Using Your Scale	5
Pull-Tab	
Measuring	
Auto-On	
Auto-Detection	
Press-Awake	
Setup Without Bluetooth	6
Setup With Bluetooth	7
Sync With Your Scale	8
Reading Your Results	9
For Best Results	
Typical Results	10
Body Fat	
Water Weight	
BMI	
Muscle Mass	
Bone Density	
Troubleshooting	11
Cleaning, Maintenance, and Disposal	12
Technical Specifications	13
Manufacturer's Warranty	14

# Important Safety Notes

## Warnings

- Never use, or allow others to use, this unit in combination with the following medical electronic devices:
  - Medical electronic implants such as pacemakers,
  - Electronic life support systems such as an artificial heart / lung,
  - Portable electronic medical devices such as an electrocardiograph.
- This scale passes a harmless and unnoticeable electrical current through your body when taking a measurement. This electrical current is not felt while using the scale. This unit may cause the above mentioned medical devices to malfunction.
- This product is not intended for use by pregnant women.
- This product is not intended for use by infants, toddlers, and children under 10 years of age.
- Do not step on the edge of the scale while getting on or off, otherwise it may tip.
- Do not jump on scale.
- Protect scale from hard knocks, temperature fluctuations, and heat sources that are too close (e.g. Stoves, heating units).
- Do not drop scale or drop any objects on it as this may damage the sensors.
- Do not step or stand on the scale when your body and / or feet are wet. For example, after taking a shower.
- Do not step on the scale when the surface is wet. There is a danger of slipping.
- Please be aware that the measurements obtained from this body analysis scale represent an approximation only.
- Do not step on this scale if you weigh more than 400 lbs.
- Place the scale on level flooring, otherwise it may tip.
- Clean the scale with a slightly damp cloth, DO NOT immerse the scale in water or use chemical/abrasive cleaning agents.

## Introduction

Welcome to your new Weight Gurus Scale.

Being healthy is worth the effort! That's why we've designed the Weight Gurus Scale to be more than just another scale. It's a beautifully designed, fully featured body composition scale that, when desired, connects to the power and convenience of your smart phone. This lets you track your progress over time, helping you stay motivated and reach your goals.


To automatically sync your results to your smartphone, download the free Weight Gurus App for iOS (Apple) or Android (see more information on page 7).

For more product information, please visit [GreaterGoods.com](http://GreaterGoods.com)

# Scale Description

## Physical Features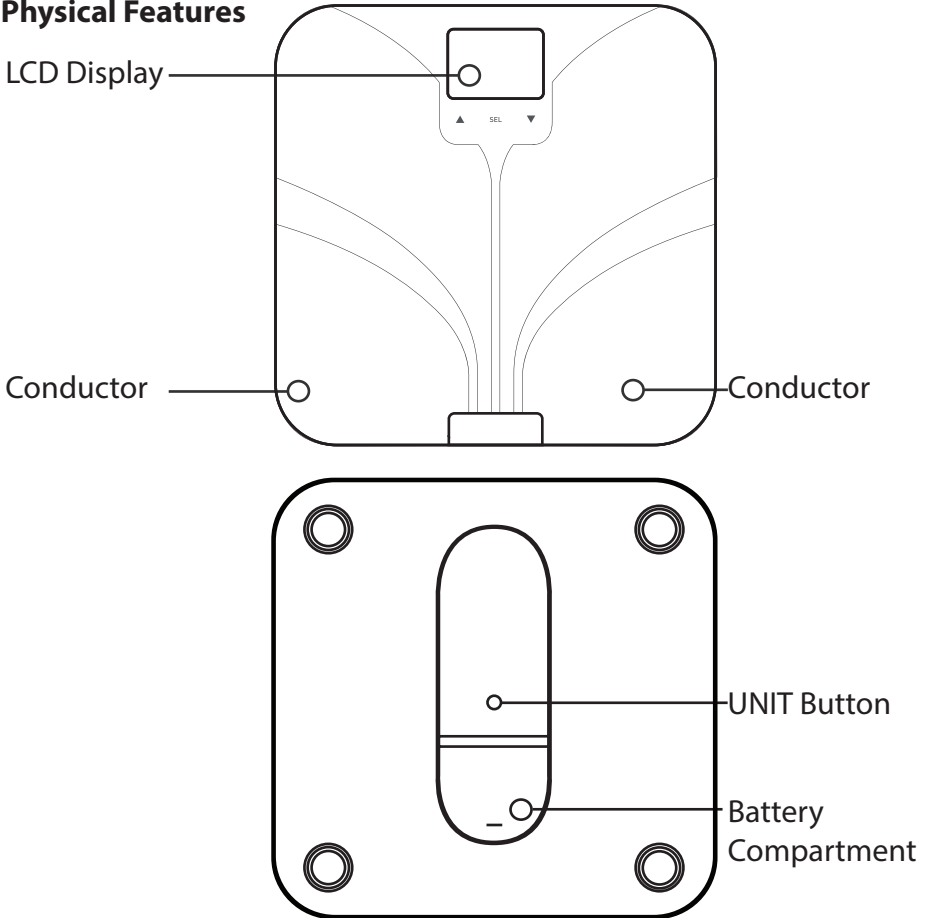

## Measuring Units

lb.	pound
kg	kilogram

## Setting The Measuring Unit

By pressing the UNIT button on the back of the scale, you can switch between lb. (pound) and kg (kilogram).

# Things To Know Before Using Your Scale

## Install Batteries

Make sure to install the four included AAA batteries before using the scale for the first time.

## Measuring

Measuring your weight and calculating body composition is easy. Simply step on your scale with bare feet, and evenly distribute your weight. Make sure that your feet are making good contact with all four conductors. Body composition can only be displayed if you have properly set up a user profile on your scale (page 6).

## Auto-On

Auto-On is the easiest way to use your scale. Simply step on the weighing surface with both feet and evenly distribute your weight. Once you complete user setup (page 6), Auto-Detection will match you to your profile automatically. If no match is made to a user profile, then the scale will only display your weight information. Body composition calculations require specific variables such as your height, gender, and activity level, so a user profile must be setup in order to display those measurements.

## Auto-Detection

Once a profile has been set up, your scale will automatically match you to your correct profile by weight. The scale determines who the user is by using a range of 10 lbs. and in some cases a conflict may arise if two or more users are within 10 lbs. of each other (see "Troubleshooting" for resolving an Auto-Detection conflict).

## Press-Awake

Another easy way to turn on your scale is to briefly step on then off the weighing area of your scale. Press-Awake allows you to enter setup mode to create or edit a new user (see User Setup, page 6). If needed, it also allows you to select your user profile manually.

# Setup Without Bluetooth

Please see page 7 for setup instructions if you plan on using Bluetooth to sync measurements to your smartphone.

**IMPORTANT: Place your scale on a hard flat surface, and be ready with bare feet to record your first weight entry on the last step.**

This scale can be customized for up to 8 users.

## Scale Only Setup (No Bluetooth Connection)

1. Press-Awake your scale on. Simply step on then off to begin.
2. Press and hold SET.
3. Use ▼ and ▲ to make selections, then tap SET to select and continue for each option below.

User Number	Choose a user profile number, tap SET			
Athlete Mode	 Male	 Male Athlete	 Female	 Female Athlete
	(See description below)			
Height	Select your height, tap SET			
Age	Select your age, tap SET			
<b>IMPORTANT: YOU MUST NOW WEIGH TO RECORD YOUR INITIAL WEIGHT ENTRY</b>				
Weight	Step on the scale with bare feet, evenly distributing your weight. Make sure that your feet are making good contact with all four conductors.			

\*At any time in the setup process, if you have to leave or the screen shuts off due to inactivity, just start over from number one above.

## Athlete Mode

An athlete is defined as a person who is involved in intense physical activity approximately 12 hours per week and who has a resting heart rate of approximately 60 beats or less per minute.

# Setup With Bluetooth

## Get The App

Search for the Weight Gurus app in your smartphone's app store. Install and register an account to get started tracking.


## Pair Your Scale With Your App

### On The App

1. Make sure your phone's Bluetooth (📶) is turned on.
2. From the Weight Gurus app menu (☰), select Bluetooth Setup.
3. Follow the setup steps within the app to finish setting up your user information and pair with your scale.

When attempting to pair, press and hold the UNIT button on the back of your scale to enter Bluetooth pairing mode. You will see the screens shown below animating back and forth.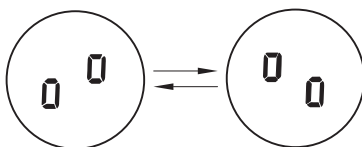

	Brackets on your scale's display indicate you have properly paired your device to your scale.
	E1 indicates an error pairing your device to your scale. Press and hold the UNIT button and try again.

### On The Scale

After pairing your scale, manually select your new profile and weigh to complete the setup process.

1. Tap SET.
2. Use ▼ and ▲ to find your user number, tap SET to select.
3. Step on your scale and weigh.

# Sync With Your Scale

## Weigh to Sync

Once setup has been completed, simply weigh to sync measurements to your device. Syncing will occur to your user account as long as your phone is within Bluetooth range and the Weight Gurus app is running.

	After weighing, the Bluetooth symbol will begin flashing as it attempts to pair to the correct device. A solid Bluetooth symbol indicates a successful connection to your device.
	The data symbol indicates measurement data is waiting to be synced to your device. Upon successfully syncing, this will go away. If measurements are not synced, it will display the next time you weigh, indicating unsynced measurements.

## Tips

Your scale will give an audible beep when it turns on, when measurements are completed, and once again when it has successfully synced to your device. If syncing cannot be completed at the time, three short beeps will indicate a failed sync. If a failure occurs, your measurements will store in the scale and attempt to sync the next time you complete weighing.


Although it's possible to view your results from many devices (such as both your mobile phone and your tablet), only one device can pair and sync to a scale for each user. Below are a few examples to further explain how pairing may work for you.

## Pairing Examples

Multiple devices can pair with one scale - e.g. Husband and wife with separate phones	Yes
Multiple users can share one device and pair to one scale - e.g. Husband and wife sharing one tablet	Yes
One user can pair to many scales - e.g. You own more than one Weight Gurus Bluetooth scale	Yes
One user can pair to one scale from multiple devices - e.g. You cannot pair both your phone and tablet to the same scale.	No

## Reading Your Results

Once you have properly weighed yourself, the scale will cycle through the measurements it has calculated for you.

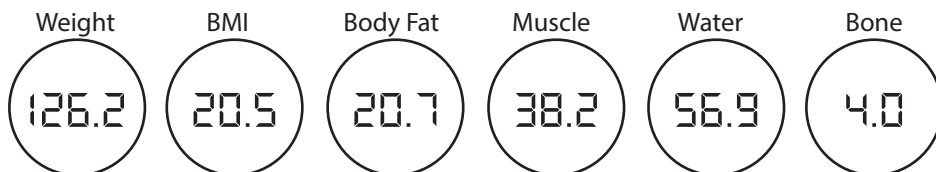### For Best Results

Remember to place your scale on a hard level surface. Step on the scale with bare feet, evenly distributing your weight. Make sure that your bare feet are making good contact with all four conductors.

Using this scale on carpet will not give accurate results.

Wearing socks during weighing will result in incorrect body composition measurements.

It is important to follow a routine. We recommend weighing daily around the same time each day. Alternatively, you can weigh weekly. When weighing weekly, we recommend weighing on the same day each week.

Weighing under the same conditions will help ensure accurate results. Weight can vary due to different clothing selections, weighing at different times of day, weighing before or after meals, etc. Following a routine will help achieve accurate results.

# Typical Results

Keep in mind that you know your body best. The following ranges of Body Fat %, Water Weight %, BMI, Muscle Mass %, and Bone Mass % are offered only as guides.

The best way to understand your body composition is to watch for changes. For example, watch to see how your body fat is going up or down over time.

## Body Fat %

Age	Female				Male			
	Low	Normal	Above Avg.	High	Low	Normal	Above Avg.	High
20-29	<19	20-28	29-31	>31	<13	14-20	21-23	>23
30-39	<20	21-29	30-32	>32	<14	15-21	22-24	>24
40-49	<21	22-30	31-33	>33	<16	17-23	24-26	>26
50-59	<22	23-31	32-34	>34	<17	18-24	25-27	>27
60+	<23	24-32	33-35	>35	<18	19-25	26-28	>28

## Water Weight %

Age	Body Fat %	Water %
Male	4 - 14%	63 - 70%
	15 - 21%	57 - 63%
	22 - 24%	55 - 57%
	25 and over	37 - 55%
Female	4 - 20%	58 - 70%
	21 - 29%	52 - 58%
	30 - 32%	49 - 52%
	33 and over	37 - 49%

## BMI

BMI	Weight Status
<18.5	Underweight
18.5 - 24.9	Normal
25 - 29.9	Overweight
30+	Obese

## Muscle Mass %

Female	Approx. 30%
Male	Approx. 40%

## Bone Mass %

	Female			Male		
	<100	100-135	>135	<135	135-165	>165
Weight	<100	100-135	>135	<135	135-165	>165
Bone	3.9	4.1	4	4	4.4	4.2

# Troubleshooting

## Low Battery

 A low battery symbol is displayed when the batteries need to be replaced. Open the battery door on the back of the scale, and remove and replace the batteries with four AAA batteries.

## Overload

— — — — Dashes are displayed when overload occurs and the scale has too much weight (Capacity: 180kg / 397lb). Please remove the weight from the scale to protect the scale sensors.

## Measuring Error

If the scale has trouble measuring your weight, it will turn off. This is most likely due to not settling on the scale fast enough. Make sure that the scale is placed on a hard flat surface (not carpet) and that you distribute your weight evenly while holding still.

## Failed To Auto-Detect Your User Profile While Weighing

Only the weight data is shown and the scale will turn off after a few seconds. Please manually select your number and measure once again (Press-Awake, select user number, Tap SET, then weigh).

## Resolving An Auto-Detection Conflict

An Auto-Detection conflict will occur if two or more users are within 10lbs of each other. When an Auto-Detection conflict occurs, your scale will give you a chance to select your profile number manually. Use the arrows to select your profile, tap SET, and step on the scale. The scale will now proceed measuring for the profile you have selected.

## How Do I Unpair A Device From My Scale?

If you no longer want to sync measurements to your device, first remove the paired scale from the app under the Scale Setup page. Next, clear your user profile from the scale itself by following the instructions below.

## How Do I Clear A User Profile On My Scale?

First, Press-Awake the scale on, then press and hold SET. Now, select the user to delete and press SET. Once selected, press the up arrow to change Clr to "Y." Now press SET to clear the user. The user is now removed.

# Cleaning, Maintenance, and Disposal

## Cleaning

Your scale is not waterproof. Never immerse the scale in water or place it in a dishwasher. To clean the scale platform, use a soft, slightly damp cloth or sponge. For stains or other residue, use a mild dish washing soap. Never use harsh cleaners or treated cloths.

## Changing The Batteries

Your scale uses 4 AAA batteries. To change the batteries, open the battery door on the bottom of the scale, remove the old batteries, and replace them with the + end oriented as shown in the battery compartment.

## Disposal

Batteries should be disposed of separately from household waste. Always dispose of batteries according to your local regulations. Dispose of this device in accordance with your local regulations.

## Technical Specifications

Weight unit: lb. / kg

Capacity: 397lb. / 180kg

Division: 0.1lb / 0.1kg

Minimal weight: 5kg

Function keys: ▼ SET ▲

Display: Positive LCD, Backlight

Batteries: 4 AAA

User No.: 8

Age range: 10 - 85

Stature range: 3ft 3.5inches to 7ft 2.5inches / 100-220cm

Measurements calculate: Body weight, BMI, body fat, water weight, muscle mass, and bone mass

Working temperature: 32°F-104°F / 0°C-40°C

Working humidity: ≤90% RH

Storage temperature: -4°F - 140°F / -20°C - 60°C

## Manufacturer's Warranty

Your scale is warranted by the manufacturer against defects in materials and workmanship for five (5) years from the original purchaser from the date of purchase. Proof of purchase is required.

The warranty is void if the product has been subjected to mechanical damage or mistreatment, such as immersion. This warranty is in lieu of all other warranties, and limits the liability of the manufacturer. This warranty gives you certain legal rights and you may have other rights depending on the state in which the product was purchased.

If your scale is defective, please contact **Greater Goods, LLC**.

**Phone:** (866) 991-8494

**Email:** [info@greatergoods.com](mailto:info@greatergoods.com)

**Web:** [GreaterGoods.com](http://GreaterGoods.com)

Greater Goods, LLC  
125 N. Main St.  
Suite 202  
St. Charles, MO 63301


# WEIGHT GURUS

---

## Báscula Inteligente con Bluetooth

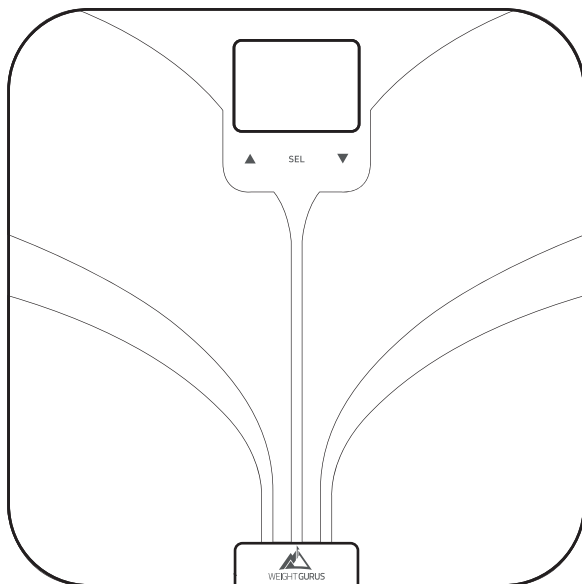---

**Get Started:** [greatergoods.com/start](http://greatergoods.com/start)

---

CUSTOMER SUPPORT

**Email:** [info@greatergoods.com](mailto:info@greatergoods.com)

**Phone:** (866) 991-8494

**Website:** [greatergoods.com](http://greatergoods.com)

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Si votre balance présente un défaut, merci de contacter **Greater Goods, LLC.**